



<http://www.unisis.co.jp>

UNIEVER[®]

Uniever is a brand name of Unisis Corp.

DISPOSABLE COMBINED SPINAL EPIDURAL ANESTHESIA MINITRAY



DISPOSABLE COMBINED SPINAL EPIDURAL ANESTHESIA MINITRAY

For the physician who will start on the procedure of CSEA

Unisis CSEA Minitray enables physicians to perform the procedure of CSEA with single puncture. The kit has simple construction yet contains reliable Epidural and Spinal anesthesia needles receiving high evaluation as quality products. Unisis CSEA Minitray is the one to be chosen as satisfactory entry-model for any physician who is thinking to try the procedure of CSEA.



Spinal Anesthesia Needle
Pencil Point



Spinal Anesthesia Needle
K-3 Lancet Point

Characteristics

●Catheter for Epidural Anesthesia

〈Biocompatibility〉

Polyurethane catheter has excellent biocompatibility to have temperature sensitive hardness, being hard enough at insertion into body. will be softened by body temperature and lowers the risk of breakage.

〈Ideal design at the tip〉

Closed-tip with helically arranged tri-holes enables smooth insertion and prevents lopsided anesthesia.



●Spinal Anesthesia Needle

Appropriate protrusion length of Spinal Needle out from Epidural Needle varies in 2 types, 13mm, and 14mm. As for the tip of the Spinal Needle, we offer both K-3 Lancet Point and less-invasive Pencil Point.



Pencil Point




K-3 Lancet Point

Description

●Epidural Needle

〈Center Pass-Hole Type〉

Back-Eye(Pass Hole) is ideally located to allow smooth insertion of Spinal Needle even at the first time of usage.

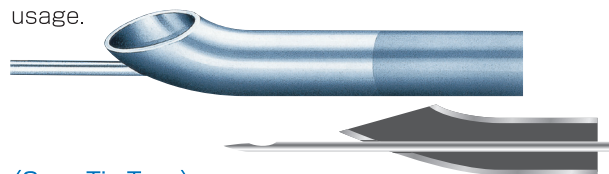


〈Open-Tip Type〉

Standard epidural needle is adopted for CSEA use.



Spinal anesthesia preceding to Epidural anesthesia through catheter enables fast and sufficient numbing effect, compared to single Epidural anesthesia.

Kit Components

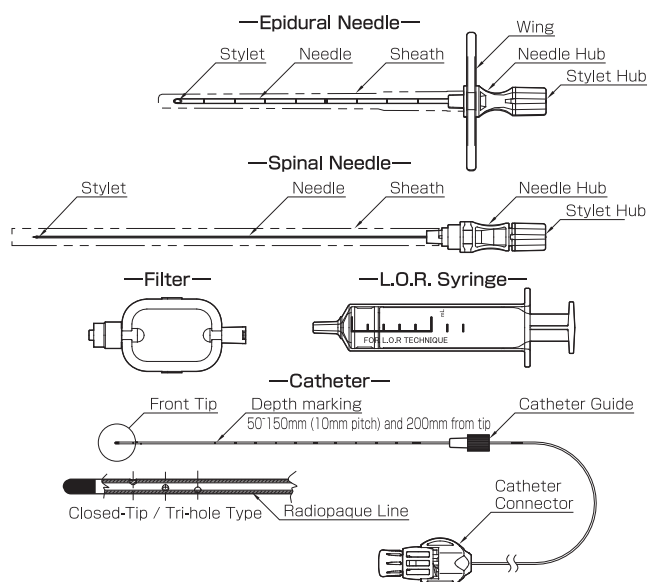

Standard sizes

Product Name : UNIEVER DISPOSABLE COMBINED
SPINAL EPIDURAL ANESTHESIA MINITRAY

G:Gauge/ Color Code	Epidural Anesthesia Needle with Huber Point	Catheter (20G×1000mm)	Spinal Anesthesia Needle	Protrusion Length of Spinal Needle	Others	REF (Code No.)
18G	90mm (with Back-Eye)	Closed-Tip with 3 lateral eyes	Pencil Point 27G x128mm	Max 14mm	·Catheter Connector ·Epidural Filter ·Catheter Guide ·L.O.R. Syringe (10mL)	5530061

Individually packed and 10 pcs/box
E.O.G. sterilized

Note: 17G Epidural Needle is also available.

Manufacturer



7th Fl. Mitsui-Sumitomo Bank Okachimachi-Bldg. 4-11-4 Taito, Taito-Ku,
Tokyo, Japan 110-0016
(Direct call) International Sales Dept. Tel: +81-3-5812-8832 / Fax: +81-3-5812-8831

URL <http://www.unisis.co.jp/english/>

Made in Japan

Representative